



helps healing.

Hydrofilm® Plus high MVTR transparent film dressing

4 stage guide to application

Measure the wound using the printed ruler on the peel pouch to ensure the correct size of dressing has been selected (allow for a 1 – 2cm overlap from the wound margin).

Remove the dressing from the peel pouch.

Step 1:

Grasp backing paper 1 between fingertips and remove to reveal the liner free portion of the dressing.



Step 2:

Position the dressing over the wound bed, with the pad central to the wound. Smooth the available adhesive on to the skin.



Step 3:

Remove backing paper 2, smoothing the dressing in place.



Step 4:

Lift the tab of backing paper 3 and remove it, following the direction of the arrow. Repeat for backing paper 4.

