



helps healing.

# Hydrofilm® Plus transparent film dressing with pad


## Guide to simple application

Measure the wound using the printed ruler on the peel pouch to ensure the correct size of dressing has been selected (allow for a 1 – 2cm overlap from the wound margin).

Remove the dressing from the peel pouch.

### Step 1:

Grasp backing paper 1 and 2 between finger tips to expose the central part of the dressing.

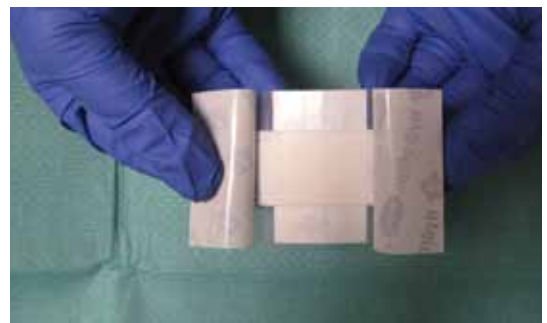

### Step 2:

Position the dressing over the wound bed, with the pad central to the wound.

Apply the dressing to the skin while removing the backing papers marked 1 and 2 using a rolling motion.

Smooth the dressing down applying light pressure.



### Step 3:

Lift the tab of backing paper 3 and remove it, following the direction of the arrow.



### Step 4:

Lift the tab of backing paper 4 and remove it, following the direction of the arrow.

