



helps healing.

Hydrofilm® high MVTR transparent film dressing


Guide to simple application

Measure the wound using the printed ruler on the peel pouch to ensure the correct size of dressing has been selected (allow for a 1 – 2cm overlap from the wound margin)

Remove the dressing from the peel pouch

Step 1:

Grasp backing paper 1 and 2 between finger tips to expose the central part of the dressing.

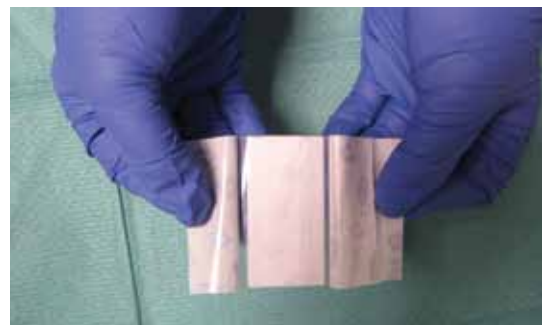

Step 2:

Position the dressing over the wound bed and apply the dressing to the skin while removing the backing papers marked 1 and 2 using a rolling motion.



Smooth the dressing down applying light pressure.

Step 3:

Lift the tab of backing paper 3 and remove it, following the direction of the arrow.



Step 4:

Lift the tab of backing paper 4 and remove it, following the direction of the arrow.

