



4-point guide for correct fitting of MoliCare® soft all-in-one briefs

1

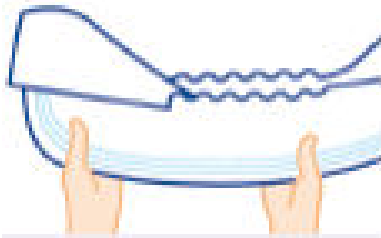


- Measure the waist and hips - use the larger size.
- Choose correct product size - if between sizes try the smaller size first.
- Open up the brief lengthways, leaving the non-woven side panels in folded position.
- Hold brief at each end and gently fold it lengthways once. This activates the inner cuffs and creates an anatomical 'cupped' shape for fitting.


2

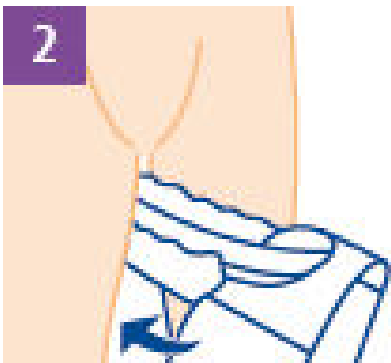

- Place the 'cupped' brief between the thighs from the front - with the absorbent and completely white surface next to the skin, the absorbent area with tabs to the rear.
- Unfold rear absorbent area and fit snugly to the body.
- Wetness indicator should be in line with the spine. At the front, centre the absorbent cushion in the crotch area.

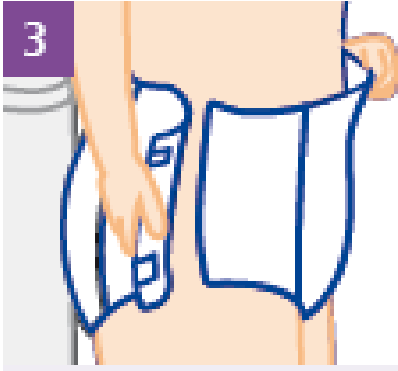

- **Unfold front area and fit into position, as high as possible in the groin area.**
- **Unfold front side panels, fit snugly to the body and unfold side panels at the back.**
- **To fasten the brief, position the upper hook and loop tabs at a slightly downwards angle onto the non-woven side panel or textile backsheet.**
- **The lower hook and loop tabs should point diagonally upwards.**

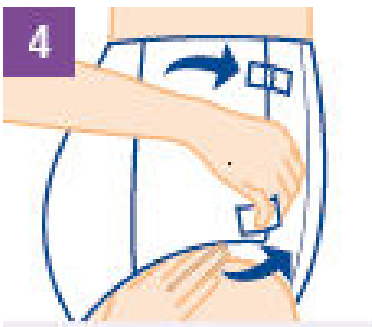

- **Check that the brief is comfortably and closely fitted.**
- **When the stripes on the product are aligned, the super absorbent core is correctly positioned.**
- **To open and reclose the tabs, open the blue stripe and reclose onto the textile backsheet or non-woven side panel.**
- **You can open and close the hook and loop tabs again and again.**



MoliCare® soft all-in-one briefs are designed for clients with severe urinary and/or faecal incontinence. The Active Skin Protection features of MoliCare® soft help to maintain the skin healthy. The hook and loop ('velcro') system allows repeated opening and closure of the tabs.

The use of oil-based or silicone-based creams, or talcum powder on the skin, may adversely affect the permeability of absorbent continence care products.

**PAUL HARTMANN LTD, Heywood Distribution Park,
Pilsworth Road, Heywood, Lancashire OL10 2TT**

Tel 01706 363200

Fax 01706 363201

Web www.hartmann.co.uk

E-mail info@uk.hartmann.info