



helps healing.

### 4-point guide for correct fitting of MoliForm® soft shaped pads



- Choose close-fitting MoliPants® (measure waist size in cms) or suitable equivalent.
- Pull MoliPants® as high up between the legs as possible, rolling the waistband downwards at the same time to form a platform between the thighs.



- Ensure MoliForm® soft is 'cupped'.
- Make a firm fold or crease along the pad lengthways ensuring the cuffs are angled inwards.
- The smaller absorbent area of MoliForm® soft should be positioned at the front of the body.

---

3



- Insert MoliForm® soft between the thighs from the front, with the absorbent and completely white surface next to the skin and the larger area for containment at the back.
- It is very important that the rear absorbent area is completely unfolded.
- The wetness indicator should be in line with the spine.

---

4



- Pull MoliPants® right up at the back, then carefully unfold the front absorbent area and pull up the net pants at the front.
  - Ensure that the textile backing is not folded and check that the net pants and the pad fit snugly.
  - Ensure the leg openings are comfortably positioned and are not pinching the skin.
-

## **MoliForm® soft**

**The use of oil-based or silicone-based creams, or talcum powder on the skin, may adversely affect the permeability of absorbent continence care products.**



**PAUL HARTMANN LTD, Heywood Distribution Park,  
Pilsworth Road, Heywood, Lancashire OL10 2TT**

**Tel 01706 363200**

**Fax 01706 363201**

**Web [www.hartmann.co.uk](http://www.hartmann.co.uk)**

**E-mail [info@uk.hartmann.info](mailto:info@uk.hartmann.info)**