





helps healing.

## 4-point guide for correct fitting of MoliForm® soft shaped pads

- Choose close-fitting **MoliPants®** (measure waist size in cms) or suitable equivalent.
- Pull **MoliPants®** as high up between the legs as possible, rolling the waistband downwards at the same time to form a platform between the thighs.

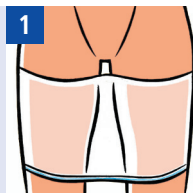

- Ensure **MoliForm® soft** is 'cupped'.
- Make a firm fold or crease along the pad lengthways ensuring the cuffs are angled inwards.
- The smaller absorbent area of **MoliForm® soft** should be positioned at the front of the body.

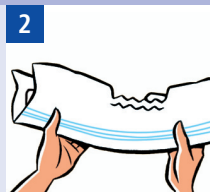

- Insert **MoliForm® soft** between the thighs from the front, with the absorbent and completely white surface next to the skin and the larger area for containment at the back.
- It is very important that the rear absorbent area is completely unfolded.
- The wetness indicator should be in line with the spine.



- Pull **MoliPants®** right up at the back, then carefully unfold the front absorbent area and pull up the net pants at the front.
- Ensure that the textile backing is not folded and check that the net pants and the pad fit snugly.
- Ensure the leg openings are comfortably positioned and are not pinching the skin.



# MoliForm® soft



The use of oil-based or silicone-based creams, or talcum powder on the skin, may adversely affect the permeability of absorbent continence care products.

Absorbency	Max. Working Capacity	Case Quantity	Product Code
------------	-----------------------	---------------	--------------

## MoliForm® soft

Light	470 ml	4 x 28	168 118
Normal	550 ml	4 x 28	168 018
Plus	720 ml	4 x 28	168 218
Extra	960 ml	4 x 28	168 318
Maxi	1110 ml	4 x 28	168 718
Super	1360 ml	4 x 28	168 918
For men	1000 ml	4 x 28	168 818

# MoliPants®



## MoliPants® soft

S	45-80 cm	5 x 60	947 795
M	60-100 cm	5 x 40	947 796
L	80-120 cm	5 x 40	947 797
XL	100-160 cm	5 x 40	947 798
XXL	140-180 cm	5 x 40	947 799



## MoliPants® 2000

M	60-100 cm	5 x 100	915 035
L	80-120 cm	5 x 100	915 036
XL	100-160 cm	5 x 100	915 044
XXL	140-180 cm	5 x 100	915 045



## MoliPants® Comfort

S	40-80 cm	5 x 10	947 759
M	60-100 cm	5 x 10	947 760
L	80-120 cm	5 x 10	947 761
XL	100-160 cm	5 x 10	947 762



## Contisure Stretch Pants

S	up to 80 cm	5 x 10	913 0123
M	60-90 cm	5 x 10	913 0124
L	80-110 cm	5 x 10	913 0125
XL	100-140 cm	5 x 10	913 0126
XXL	140cm+	5 x 10	913 0127