



## Washing instructions for P&S Healthcare range of washables

The P&S range of 100% cotton washable briefs includes discreet, built-in absorbent pads, and is suitable for slight to moderate incontinence.

In order to achieve best product performance we highly recommend adhering to the following washing instructions for the P&S washables range.

**PLEASE WASH ALL PRODUCTS BEFORE FIRST USE FOLLOWING THE WASHING INSTRUCTIONS BELOW:**

### Bodyworn products

- Machine-wash at 40°C
- Can be washed up to 200 times
- Do not use fabric softener, do not tumble dry, do not iron, dry-clean and do not use bleach on the products

### Bed pads

- Machine-wash at 70°C - 75°C \*
- Can be washed up to 200 times
- Can be tumble dried on LOW temperature
- Do not use fabric softener, do not iron, do not dry-clean and do not use bleach on the products

\* Depending on the make of your washing machine, the washing temperature that you can select may vary. 75°C is the ideal washing temperature, but if this is not available on your washing machine, select the next washing temperature below this. The products should not be washed over 75°C.

