

April 13, 2006

## Results

Fluid Handling Properties following 72 hours incubation

Dressing	Mean Moisture Vapour Loss (g/10cm <sup>2</sup> )	Mean Absorbency (g/10cm <sup>2</sup> )	Mean Fluid Handling Capacity (g/10cm <sup>2</sup> )
Permafoam	20.31 (1.100)	6.37 (0.199)	26.68 (1.144)
Allevyn Non-Adhesive	5.22(0.053)	4.51 (0.332)	9.72 (0.373)

## Note:

- Results are the mean of five replicates.
- Figures in brackets denote standard deviations.



The Surgical Materials Testing Laboratory is UKAS accredited for testing medical disposables and dressings.

Bro Morgannwg NHS Trust.  
Ymddiriedolaeth Gig Bro Morgannwg.