


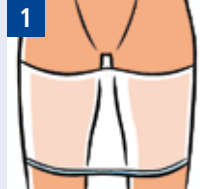

4-point guide for correct fitting of MoliForm® shaped pads

- Choose close-fitting **MoliPants®** (measure waist size in cms) or suitable equivalent.
- Pull **MoliPants®** as high up between the legs as possible, rolling the waistband downwards at the same time to form a platform between the thighs.



- Ensure **MoliForm®** is 'cupped'.
- Make a firm fold or crease along the pad lengthways ensuring the cuffs are angled inwards.
- The smaller absorbent area of **MoliForm®** should be positioned at the front of the body.



- Insert **MoliForm®** between the thighs from the front, with the soft white side next to the skin and the larger area for containment at the back.
- It is very important that the rear absorbent area is completely unfolded.
- The wetness indicator should be in line with the spine.



- Pull **MoliPants®** right up at the back, then carefully unfold the front absorbent area and pull up the net pants at the front.
- Ensure that the waterproof backing is not folded and check that the net pants and the pad fit snugly.
- Ensure the leg openings are comfortably positioned and are not pinching the skin.



MoliForm®



The use of oil-based or silicone-based creams, or talcum powder on the skin, may adversely affect the permeability of absorbent continence care products.

International Standard Method ISO 11948-1

Absorbency	ISO Absorbency	Case Quantity	Product Code
MoliForm®			
Light	1250 ml	4 x 28	168 091
Normal	1641 ml	4 x 28	168 022
Plus	1857 ml	4 x 28	168 222
Extra	2626 ml	4 x 28	168 322
Maxi	2784 ml	4 x 28	168 185
Super	3211 ml	4 x 28	168 922

MoliPants®



MoliPants® Soft

S	45-80 cm	5 x 40	947 795
M	60-100 cm	5 x 40	947 796
L	80-120 cm	5 x 40	947 797
XL	100-160 cm	5 x 40	947 798
XXL	140-180 cm	5 x 40	947 799

MoliPants® 2000

M	60-100 cm	5 x 60	947 741
L	80-120 cm	5 x 60	947 742
XL	100-160 cm	5 x 60	947 743
XXL	140-180 cm	5 x 60	947 744

MoliPants® Comfort

S	40-80 cm	5 x 60	947 734
M	60-100 cm	5 x 60	947 735
L	80-120 cm	5 x 60	947 736
XL	100-160 cm	5 x 60	947 737

Contisure Stretch Pants

S	up to 80 cm	5 x 10	913 0123
M	60-90 cm	5 x 10	913 0124
L	80-110 cm	5 x 10	913 0125
XL	100-140 cm	5 x 10	913 0126
XXL	140cm+	5 x 10	913 0127